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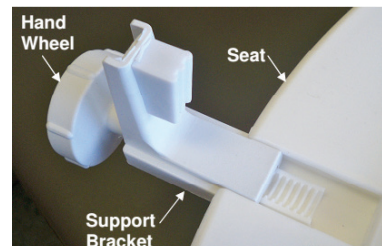
SET UP

Check all parts carefully before assembling the product. Do not assemble if any of the parts are damaged. Do not assemble if any of the parts are missing. The package must contain below parts:

- 1 x Raised toilet seat
- 2 x „L“ shape supports
- 2 x Hand wheel

If you found all parts, you may then start installation. Before installation ensure that the existing toilet seat assembly is fitted securely and undamaged. Lift the lid of the existing toilet seat prior to installation of the raised toilet seat.

1. Place the raised toilet seat with its top down onto a flat surface and install the support brackets into the slots on the seat (diagram 1)
2. Slip one of the L-shaped brackets into one of the slots in the side of the raised toilet seat. Then push lightly on the bracket and turn the knob clockwise until its thread engages. Repeat with the second bracket on the opposite side of the seat. Do not tighten any further at this stage.
3. Turn the raised toilet seat over and place it onto the existing standard toilet bowl and tighten both brackets until snug against the existing toilet bowl.

Warning!

Always ensure raised toilet seat is securely installed and fitted to the existing toilet by a competent person prior to use. Do not hold onto the seat as a support when getting on and off the toilet seat. Do not sit on top of the lid of the raised toilet seat. Inspect regularly for wear and damage as well as for secure fit.

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• We issue a warranty of 12 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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